
PROVOX[®] Adhesives

Life









Standard Adhesives
Stability Adhesive

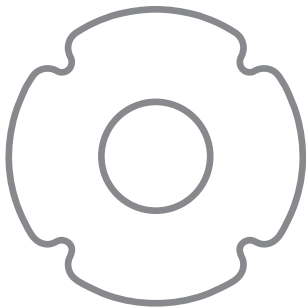





Sensitive Adhesives
Night Adhesive







Provox Life Standard Adhesives, Provox Life Sensitive Adhesives

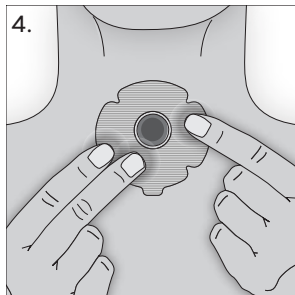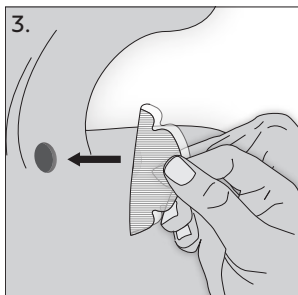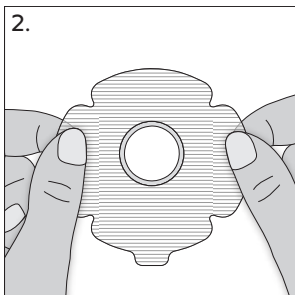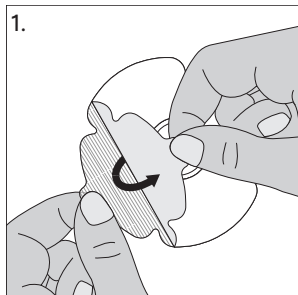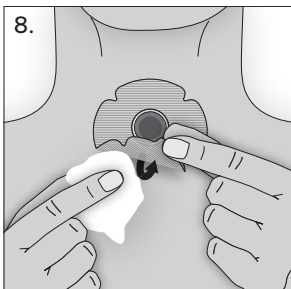
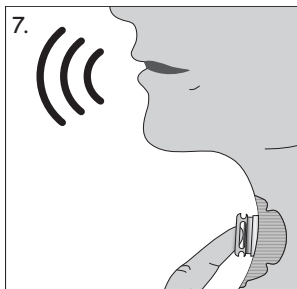
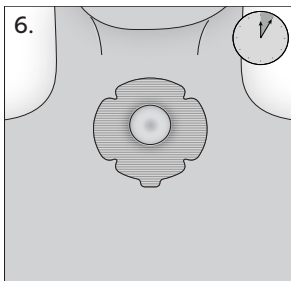
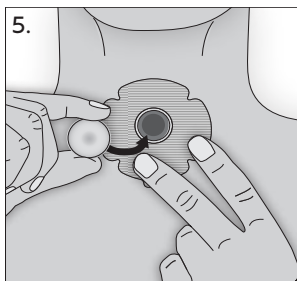








Provox Life Stability Adhesive

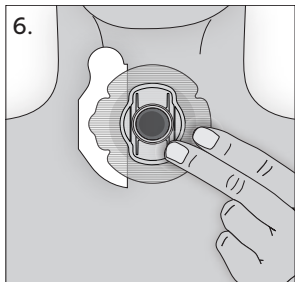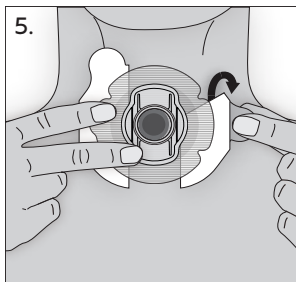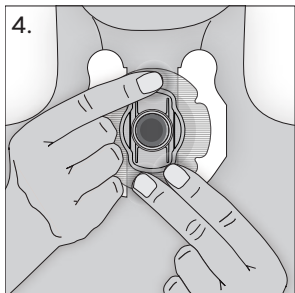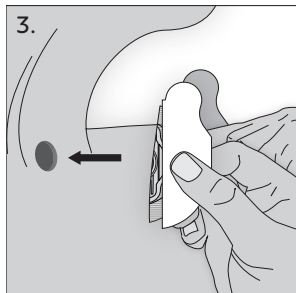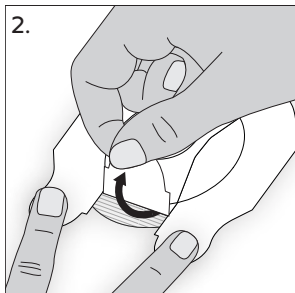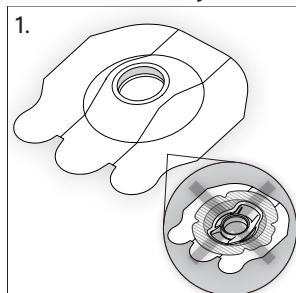


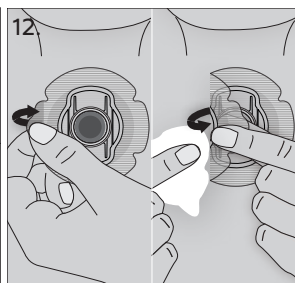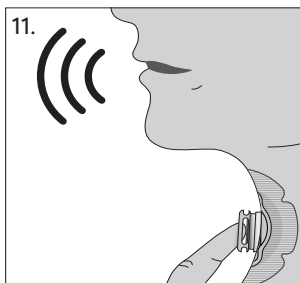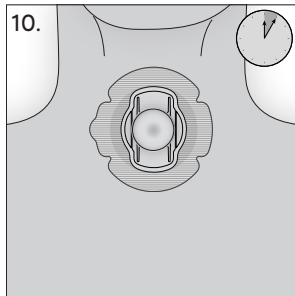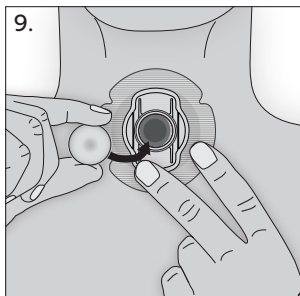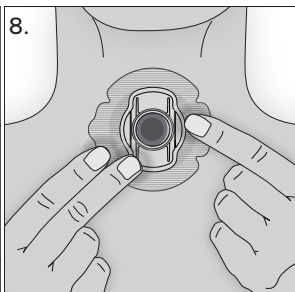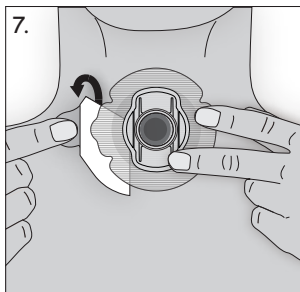
Provox Life Night Adhesive

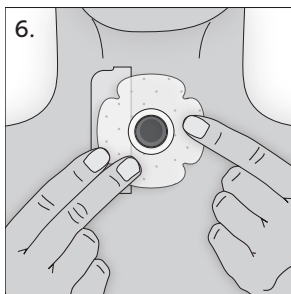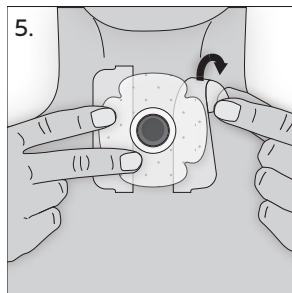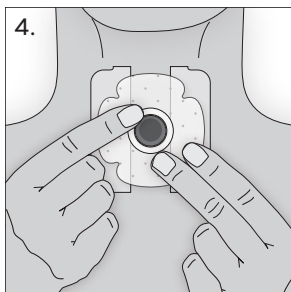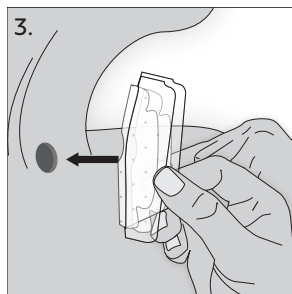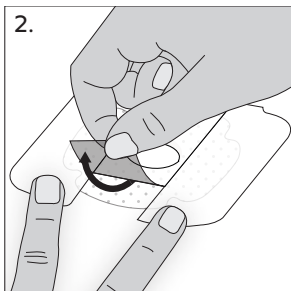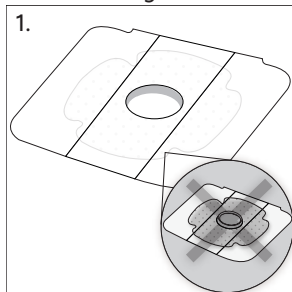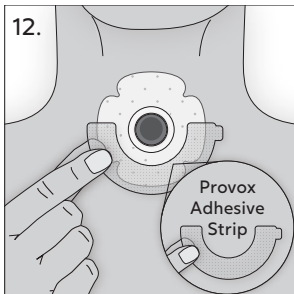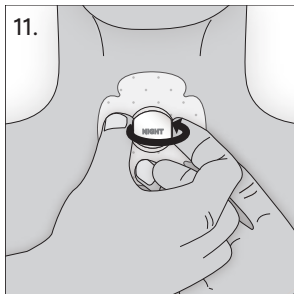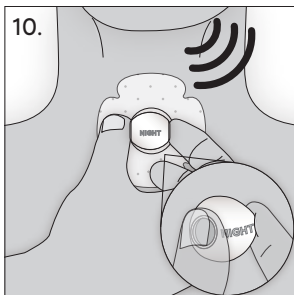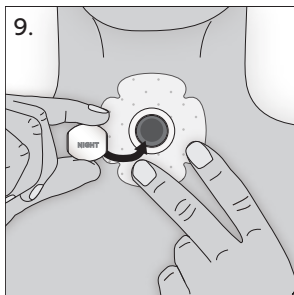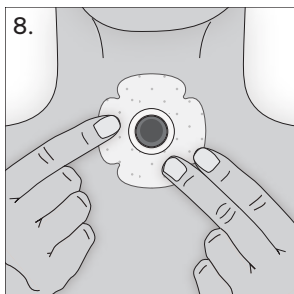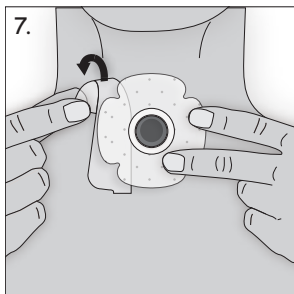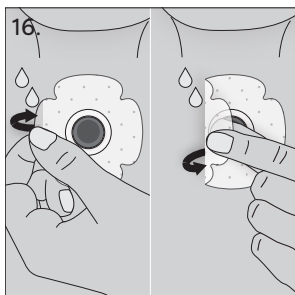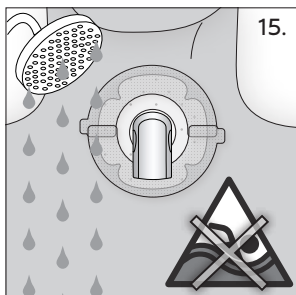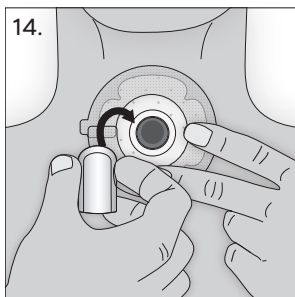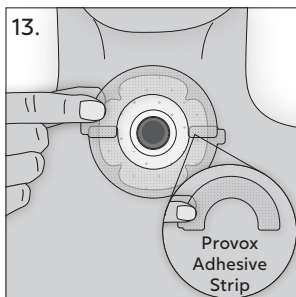Disclaimer

Atos Medical offers no warranty – neither expressed nor implied – to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

Patents and trademarks

Provox[®] is a registered trademark owned by Atos Medical AB, Sweden.

Provox[®] Life[™] is a trademark of Atos Medical AB.

For information about protective rights (e.g. patents), please refer to our webpage www.atosmedical.com/patents.

Compatible Products; Kombinierbare Produkte; Compatibele producten; Produits compatibles; Prodotti compatibili; Productos compatibles; Produtos compatíveis; Kompatibla produkter; Kompatible produkter; Yhteensopivat tuotteet; Ühilduvad tooted; Kompatibilis termékek; Zgodne produkty; Kompatibilni proizvodi; 対応製品; المنتجات المتوافقة

	Provox Life™ Standard Adhesives	Provox Life™ Sensitive Adhesives	Provox Life™ Stability Adhesive	Provox Life™ Night Adhesive
Provox Life™ HMEs	X	X	X	X
Provox Life™ Shower	X	X	X	X
Provox Life™ Baseplate Adaptor	X	X	X	X
Provox Life™ LaryTube with Ring	X	X	X	X
Provox Life™ LaryTube Fenestrated with Ring	X	X	X	X
Provox Cleaning Towel	X	X	X	X
Provox Silicone Glue	X	X	X	
Provox Skin Barrier	X	X	X	
Provox Adhesive Remover	X	X	X	

Manufacturer; Hersteller; Fabrikant; Fabricant; Produttore;
 Fabricante; Tillverkare; Producent; Produsent;
 Valmistaja; Framleiðandi; Tootja; Ražotājs; Gamintojas;
 Výrobce; Gyártó; Výrobca; Proizvajalec; Producent; Producător;
 Proizvođač; Proizvođač; Κατασκευαστής; Производител; Üretici;
 მწარმოებელი; Ἀρτοποιητής; İstehsalçı; Производитель; Pro-
 dusen; Pengilang; 製造業者; 제조사; 製造商; 制造商;

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1. Intended use

Provox Life™ Adhesives are single use adhesives that provide attachment for Provox Life™ HMEs and accessories after total laryngectomy.

2. Contraindications

The product shall not be used by patients with a decreased level of consciousness, patients with reduced mobility of the arms and/or hands, or patients who are unable to remove the device themselves.

3. Description of the device

Provox Life™ Standard Adhesive is an acrylic adhesive for everyday use. It is soft, flexible and has a low-profile to provide a strong seal and secure fit around the stoma. Provox Life™ Stability Adhesive is an acrylic, firm everyday adhesive that provides solidity, especially if a stoma is deep. It is ideal for those who speak frequently or speak hands-free.

Provox Life™ Sensitive Adhesive is made of a hydrocolloid material that is suitable for sensitive skin. For example, dependent on clinical suitability and continued tolerance, it can be used postoperatively and during and after radiotherapy.

Provox Life™ Night Adhesive is made of hydrogel material that is suitable for sensitive skin. For example, dependent on clinical suitability and continued tolerance, it can be used for overnight skin-recovery, postoperative application and use during and after radiotherapy.

4. WARNINGS

- Only use Provox Life™ Adhesives with compatible HMEs and accessories.
- Do not use Provox Life™ Standard Adhesives or Provox Life™ Stability Adhesive directly after laryngectomy.
- Provox Life™ Adhesives needs to be assessed both on an individual basis and reviewed regularly through the course of radiotherapy treatment.

5. PRECAUTIONS

- The adhesive may irritate the skin. Stop using the adhesive if skin irritation develops and consult your clinician.
- Provox Life™ Sensitive Adhesives and Provox Life™ Night Adhesive can be used on irritated skin. However, if skin irritation persists for more than 14 days, stop using the adhesive and consult your clinician.
- When cleaning the skin from residual glue, prevent particles/fluids from entering the tracheostoma.
- When taking a shower using a Provox Life™ Adhesive, use the Provox Life™ Shower (read the Instructions for Use accompanying the product).

Precautions specific for Provox Life™ Night Adhesive:

- The hydrogel adhesive material absorbs moisture from the skin and the environment (humid air, sweat etc.) and may come loose if it becomes too wet. Consult your clinician if you need an adhesive with a stronger adhesion.
- Do not use skin barrier products or products (e.g. skin cream) that leave oily residue on the skin that may prevent the adhesive from sticking properly.
- When taking a shower using a Provox Life™ Night Adhesive use the Provox Life™ Shower and Provox Adhesive Strips (read the Instructions for Use accompanying the products). The hydrogel adhesive material may absorb water and detach.

6. Instructions for use

Clean

Always clean the skin with a Provox Cleaning Towel and/or soap and water and let the skin dry before applying the adhesive.

Prepare and apply

If needed, use Provox Skin Barrier or Provox Silicone Glue (read the Instructions for Use accompanying the product) before applying the adhesive.

Apply the adhesive to the skin around the stoma. Massage the adhesive gently.

Attach an HME. Wait at least 5 minutes before speaking to improve the adhesion.

Instructions specific for Provox Life™ Night Adhesive:

Provox Skin Barrier and Provox Silicone Glue should not be used.

Remove

Gently remove the adhesive using the finger-lift tab. Provox Adhesive Remover can be used for easy removal (read the Instructions for Use accompanying the product).

Instructions specific for Provox Life™ Night Adhesive:

Gently remove Provox Life™ Night Adhesive. Water can be used to help remove the adhesive.

7. Disposal

Always follow medical practice and national requirements regarding biohazards when disposing of a used medical device.

8. Reporting

Please note that any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the national authority of the country in which the user and/or patient resides.

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