# FREEVENT<sup>®</sup>

## XtraCare<sup>™</sup>

Figure 1

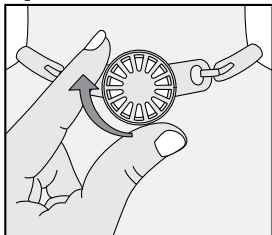
Figure 2

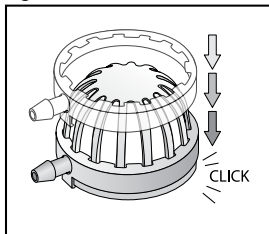
Figure 3


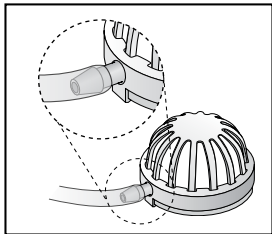
Figure 4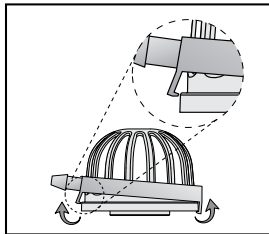

## **Prescription information**

**CAUTION:** United States Federal law restricts this device to sale, distribution and use by or on order of a physician or a licensed practitioner. The availability of this product without prescription outside the United States may vary from country to country.

## **Disclaimer**

Atos Medical offers no warranty – neither expressed nor implied – to the purchaser hereunder as to the lifetime of the product delivered, which may vary with individual use and biological conditions. Furthermore, Atos Medical offers no warranty of merchantability or fitness of the product for any particular purpose.

## **Patents and Trademarks**

Freevent® is a registered trademark owned by Atos Medical AB, Sweden. XtraCare™ is a trademark of Atos Medical AB. For information about protective rights (e.g. patents), please refer to our web page [www.atosmedical.com/patents](http://www.atosmedical.com/patents).

Manufacturer; Hersteller; Fabrikant; Fabricant; Produttore; Fabricante; Tillverkare; Producent; Produsent; Valmistaja; Framleiðandi; Tootja; Ražotājs; Gamintojas; Výrobce; Gyártó; Výrobca; Proizvajalec; Producent; Producător; Proizvođač; Proizvođač; Κατασκευαστής; Производител; Üretici; მწარმოებელი; Ἀρτοποιητής; İstehsalçı; Производитель; Produsen; Pengilang; 製造業者; 제조사; 製造商; 製造商; היצרן; الجهة المصنعة

Date of manufacture; Herstellungsdatum; Productiedatum; Date de fabrication; Data di fabbricazione; Fecha de fabricación; Data de fabrico; Tillverkningsdatum; Fremstillingsdato; Produksjonsdato; Valmistuspäivä; Framleiðsludagsetning; Valmistamise kuupäev; Ražošanas datums; Pagaminimo data; Datum výroby; Gyártás időpontja; Dátum výroby; Datum proizvodnje; Data produkci; Data fabricației; Datum proizvodnje; Datum proizvodnje; Ημερομηνία κατασκευής; Дата на производство; Üretim tarihi; წარმოების თარიღი; Ἀρτοποιημάτων ἀντιστοιχία; İstehsal tarixi; Дата изготовления; Tanggal produksi; Tarikh pembuatan; 製造日; 제조일자; 製造日期; 製造日期; תאריך ייצור; تاريخ التصنيع

Use-by date; Verfallsdatum; Houdbaarheidsdatum; Date limite d'utilisation; Data di scadenza; Fecha de caducidad; Data de validade; Används senast; Anvendes inden; Brukes innen; Viimeinen käytöpäivä; Síðasti notkunardagur; Kölblik kuni; Derīguma termiņš; Naudoti iki; Datum expirace; Felhasználatóság dátuma; Dátum najneskoršej spotreby; Rok uporabnosti; Data ważności; Data expirării; Rok uporabe; Rok upotrebe; Ημερομηνία λήξης; Срок годности; Son kullanım tarihi; ვარგისი; Ὡρισμένη ἡμερομηνία ὁλοκληρώσεως; Son istifadə tarixi; Срок годности; Gunakan sebelum tanggal; Tarikh guna sebelum; 使用期限; 사용 기한; 使用截止日期; 使用截止日期; תאריך אחרון לשימוש; تاريخ انتهاء الصلاحية

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## 1. Descriptive information

### 1.1 Intended use

Freevent XtraCare is a single use Heat and Moisture Exchanger with electrostatic filter (HMEF) that conditions and filters inhaled air in patients spontaneously breathing through a tracheostoma.

### 1.2 CONTRAINDICATIONS

Freevent XtraCare is contraindicated for patients who:

- are under any form of mechanical ventilation.
- are unable to handle or remove the device themselves when needed, and who are not under constant supervision of a clinician or a trained caregiver.
- use the device on a single lumen tube (tube without an inner tube), unless the patient or caregiver is able to reinsert the tube themselves after emergency or accidental removal.
- cannot tolerate the added dead space.

### 1.3 Description of the device

Freevent XtraCare is a Heat and Moisture Exchanger combined with an electrostatic filter (HMEF). The HME is impregnated with a hygroscopic salt and conditions the inhaled air. The electrostatic filter reduces the inhalation of particles such as viruses, bacteria, pollen and other particulate matter through the tracheostoma. NOTE: Since there are other pathways for pathogens to enter the human body, Freevent XtraCare can never guarantee total protection. Freevent XtraCare has a 15 mm ISO connector for connection to a tracheostomy tube. Freevent XtraCare can be connected to oxygen tubing using Freevent O<sub>2</sub> Adaptor. See section 3.

## 1.4 Technical data

|                                                                                          |                               |
|------------------------------------------------------------------------------------------|-------------------------------|
| Dead space                                                                               | 5.5 ml                        |
| Pressure drop* after 30 min at<br>- 15 l/min<br>- 30 l/min<br>- 60 l/min                 | 41 Pa<br>86 Pa<br>189 Pa      |
| Pressure drop* after 24 h at<br>- 15 l/min<br>- 30 l/min<br>- 60 l/min                   | 44 Pa<br>93 Pa<br>204 Pa      |
| Moisture loss* at<br>- VT= 250 ml<br>- VT= 1000 ml                                       | 12.4 mg/l<br>19.3 mg/l        |
| Filtration efficiency**<br>- Virus (VFE)<br>- Bacteria (BFE)<br>- Particles (PFE) 0.1 µm | >99.0 %<br>>99.0 %<br>>95.0 % |

\* According to ISO 9360

\*\* According to ASTM F2100

## 1.5 WARNINGS

**DO NOT** reuse Freevent XtraCare on another patient. The device and all its accessories are intended for single use only. Reuse in another patient may cause cross contamination.

## 1.6 PRECAUTIONS

- Do not use a damaged or disassembled Freevent XtraCare as this may cause aspiration of small parts.
- Do not use a soiled or contaminated device as this may cause an infection.
- Do not wash and re-use the HME. Washing the HME impairs the filtering- and HME functions.
- Do not use the device beyond 24 hours after initial use. This can increase the risk for infection due to bacterial colonization.
- Do not administer medicated nebulizer treatment over the device since the medication can become deposited in the device.
- Do not use humidifiers or heated humidified oxygen over the device since the HME will become too wet.

## 2. Instructions for use

The device is for single use and must be replaced at least every 24 hours, or more often as needed (e.g. if clogged by mucus or wet).

To use Freevent XtraCare, gently attach the device to the tube using a clockwise twisting motion (Fig. 1). To remove the device from the tracheostomy tube, gently pull the device using a clockwise twisting motion.

### 2.1 Disposal

Always follow medical practice and national requirements regarding biohazard when disposing of a used medical device.

## 3. Accessories

### Freevent O<sub>2</sub> Adaptor:

Freevent O<sub>2</sub> Adaptor is an accessory for patients requiring supplemental oxygen. The adaptor connects oxygen tubing of 1/8 inch (3.2 mm) diameter to the oxygen port. **CAUTION:** The adaptor is for single use and should be replaced when it becomes dirty, contaminated or shows any signs of damage.

Attach and detach the O<sub>2</sub> Adaptor to the device as shown in Fig. 2 and Fig. 4. Attach the oxygen tubing to the adaptor as shown in Fig. 3. Make sure the oxygen tube goes beyond the conical edge of the connector (Fig. 3).

For ordering information, see end of this Instructions for Use. For additional help or information, please see back cover of the Instructions for Use for contact information.

## 4. Reporting

Please note that any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the national authority of the country in which the user and/or patient resides.

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